



Loading instruction – Ergotouch Fly Monofocal LENTIS® with C-Loop-haptic



L-302-1
L-312
LS-312Y

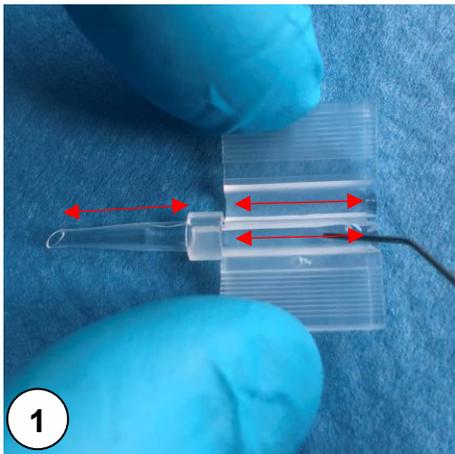


1
Lubricate the injector tip + loading chamber with viscoelastic solution.

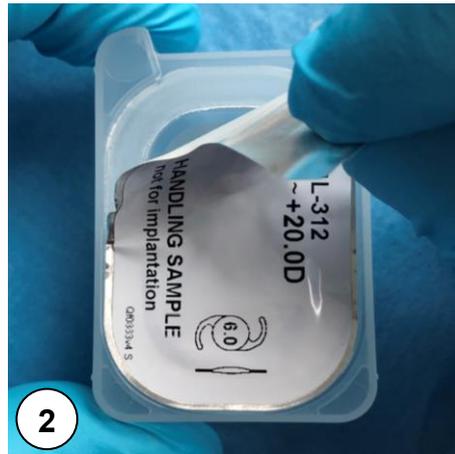


2
Open the IOL blister **to your body**.




3
Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

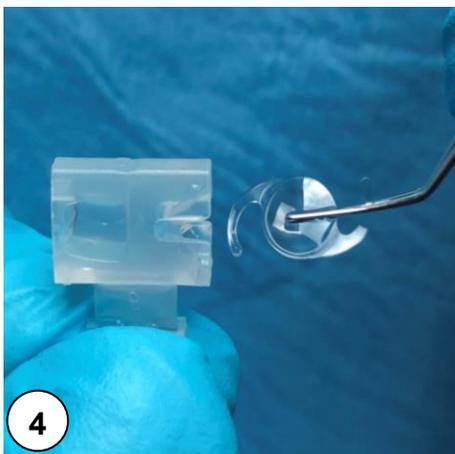

4
Use a non-toothed forceps to remove the IOL from the holder without excessive force & keep the **notch face to the right side!**

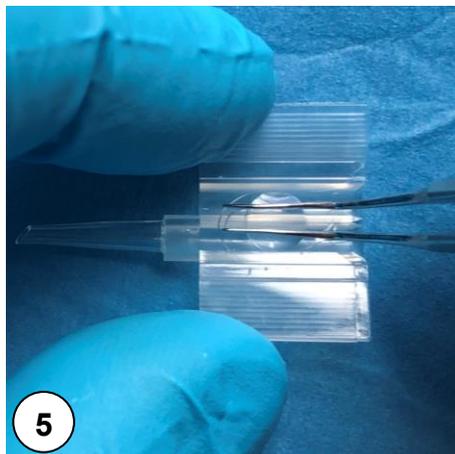

5
Place the IOL within the cartridge with the **correct haptic alignment**. **Push the IOL down** with forceps while closing the cartridge, and a soft **“click”** shall be noticed.

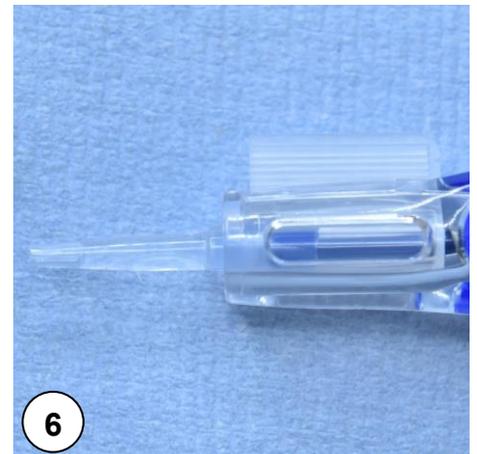

6
Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

