


Loading instruction - Viscoject Bio Monofocal LENTIS® with C-Loop-haptic




L-302-1
L-312
LS-312Y

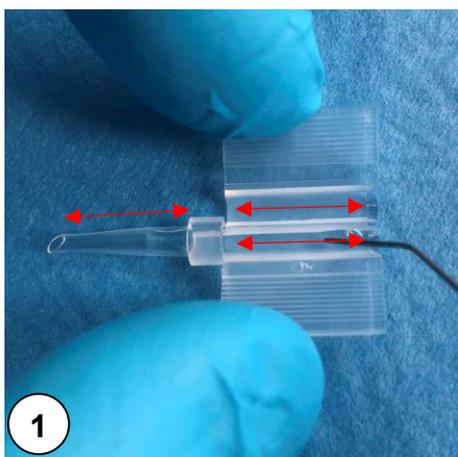

1

Lubricate the injector tip + loading chamber with viscoelastic solution.

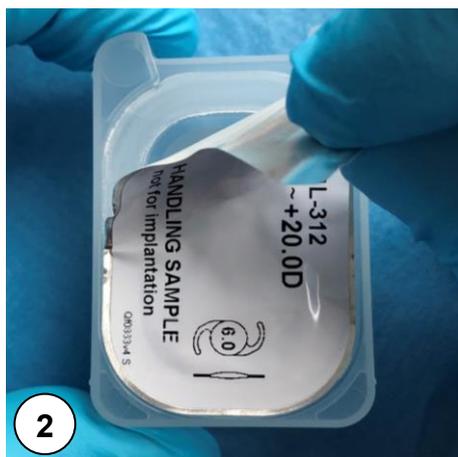


2

Open the IOL blister **to your body**.




3

Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

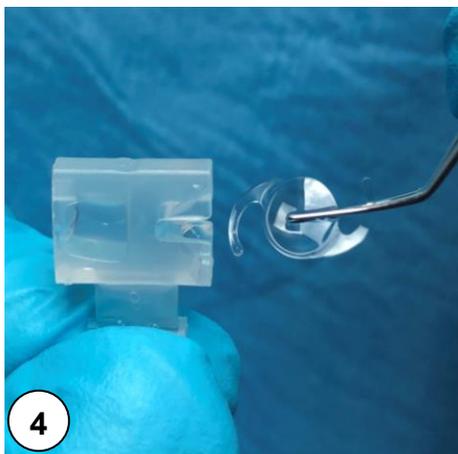


4

Use a non-toothed forceps to remove the IOL from the holder without excessive force & keep the **notch face to the right side!**

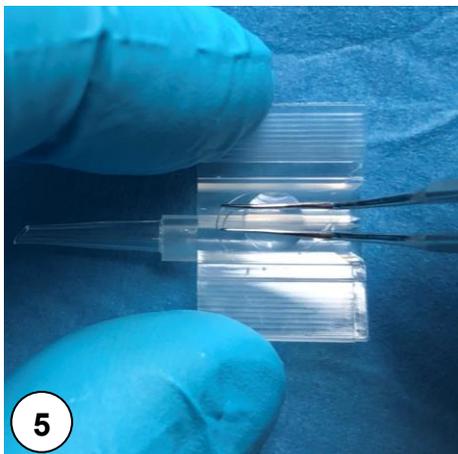



5

Place the IOL within the cartridge, ensure the **correct haptic alignment**. **Push the IOL down with forceps** while closing the cartridge, and a soft **“click”** shall be noticed.

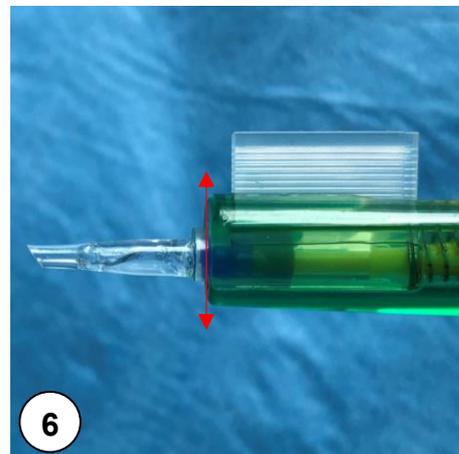

6

Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

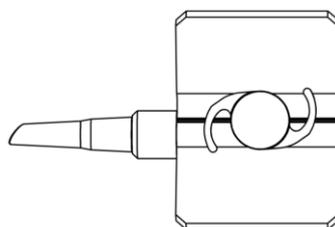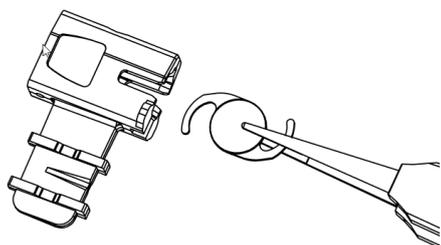

Scan for Video



Teleon Surgical B.V. | Van Rensselaerweg 4 b | 6956AV Spankeren | The Netherlands

www.teleon-surgical.com | iol@teleon-surgical.com

QF2304Av05