



Loading instruction - Viscoject Bio


Monofocal LENTIS® with plate-haptic

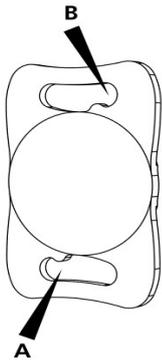

L-303
L-313
LS-313Y
L-323
LU-323
L-333

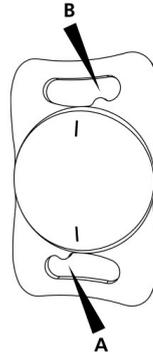

LS-313 T0-6
LU-313 T
LU-313 TY
LU-323 T
LU-323 TY

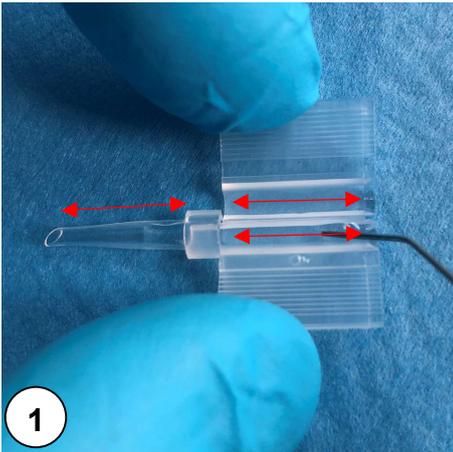


1
Lubricate the injector tip + loading chamber with viscoelastic solution.




2
Open the IOL blister away from your body.




3
Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

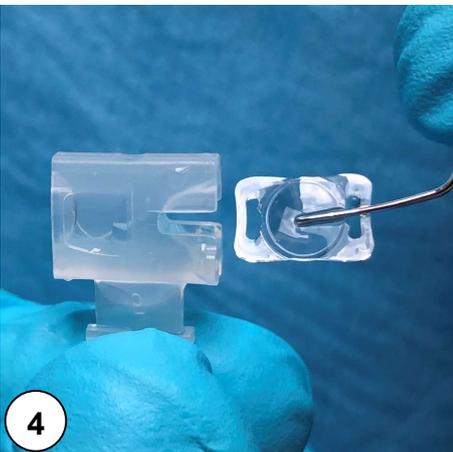

4
Notch should face to the right side! Remove the IOL from the holder with forceps without excessive force. **Use a non-toothed forceps!**

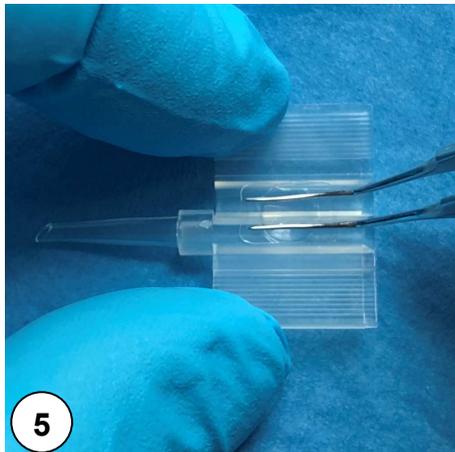

5
Place the IOL within the cartridge. Pay attention to marking A and B. Push the IOL down with forceps while closing the cartridge.

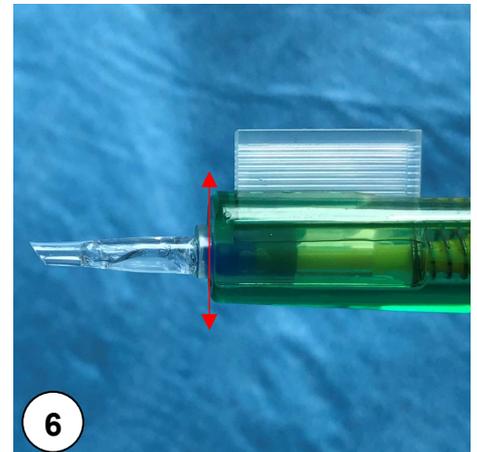

6
Place the cartridge into the injector. Push the plunger until the blue cushion reaches the tip, followed by short pressure relief. Implant the IOL.

