

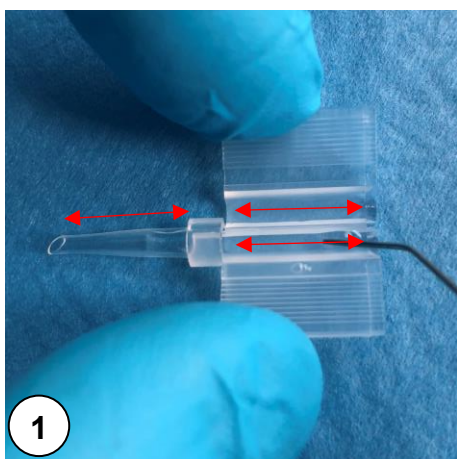
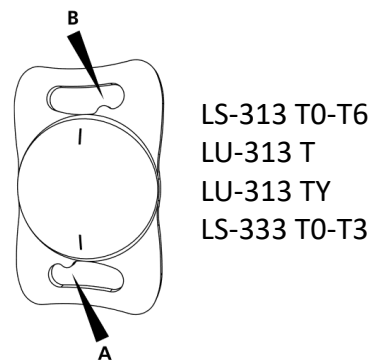
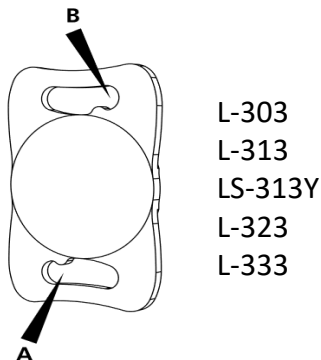


Loading instruction - Viscoject Bio Monofocal LENTIS® with plate-haptic

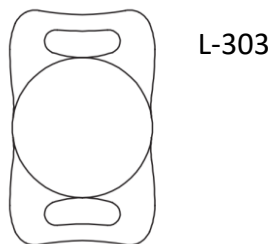


Lubricate the injector tip + loading chamber with viscoelastic solution.

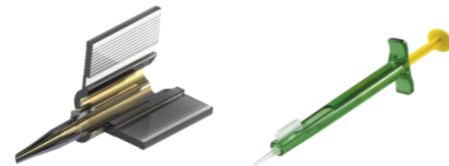


Open the IOL blister **to your body**.




Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

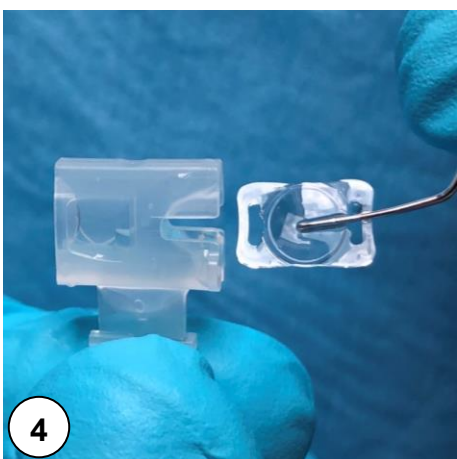



Use a non-toothed forceps to remove the IOL from the holder without excessive force & keep the **notch face to the right side!**

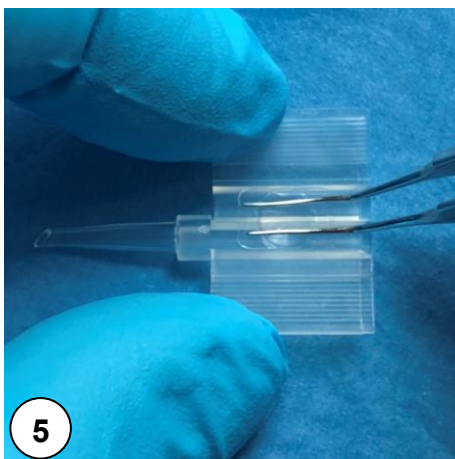

Place the IOL within the cartridge, ensure the **correct haptic alignment of A and B**. **Push the IOL down** with forceps while closing the cartridge, and a soft **"click"** shall be noticed.

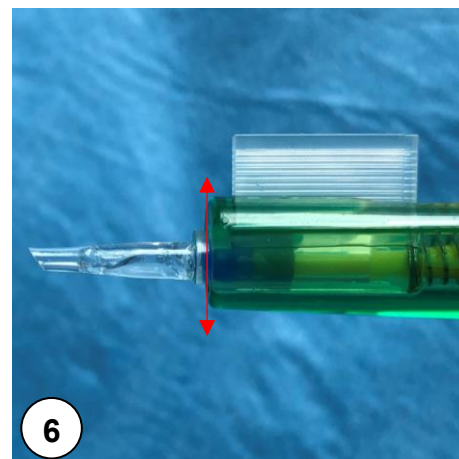

Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

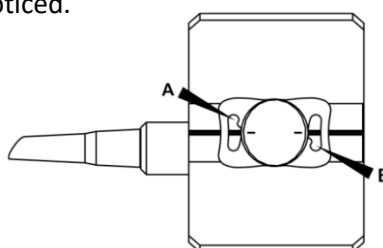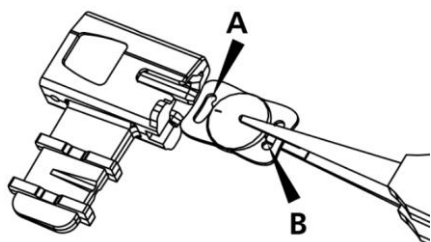

Scan for Video

