




Loading instruction - Viscoject Bio FEMTIS® Family with plate-haptic

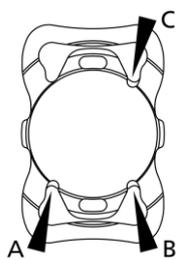


FB-313

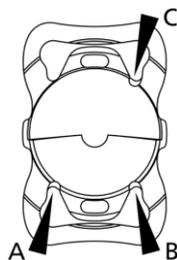

FB-313 MF15
FB-313 MF30

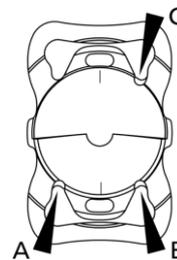

FB-313 MF15 TO-T3
FB-313 MF30 TO-T3

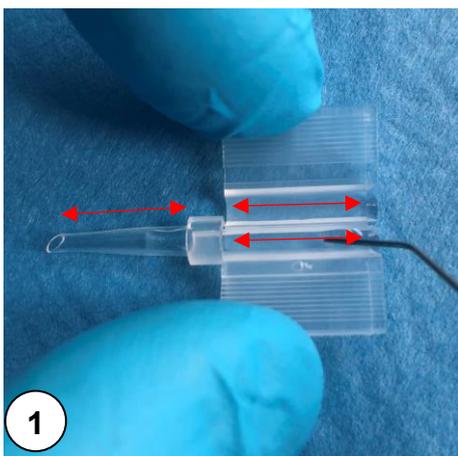


1 Lubricate the injector tip + loading chamber with viscoelastic solution.




2 Open the IOL blister **to your body**.




3 Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

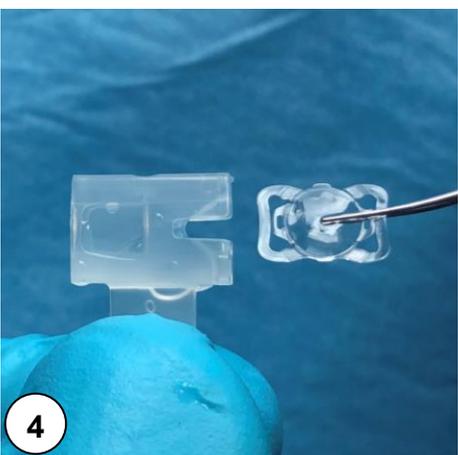



4 **Use a non-toothed forceps** to remove the IOL from the holder without excessive force & keep the **notch face to the right side!**

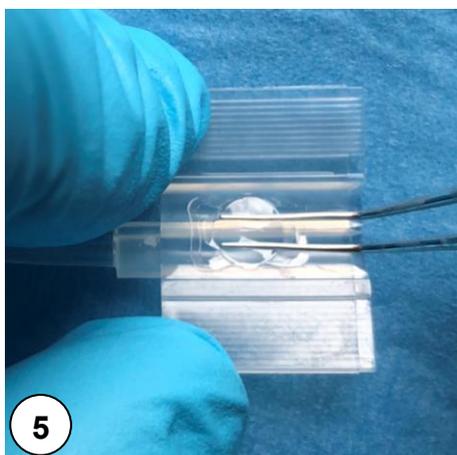

5 Place the IOL within the cartridge, ensure the **correct haptic alignment of A, B and C**. **Push the IOL down** with forceps while closing the cartridge, and a soft **"click"** shall be noticed.

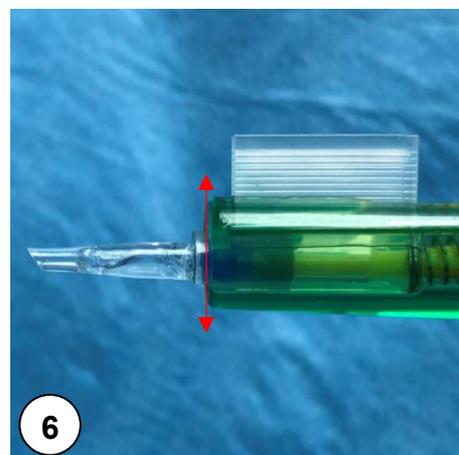

6 Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

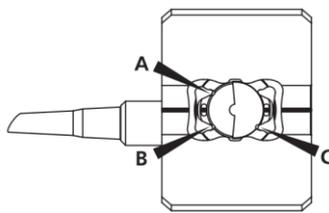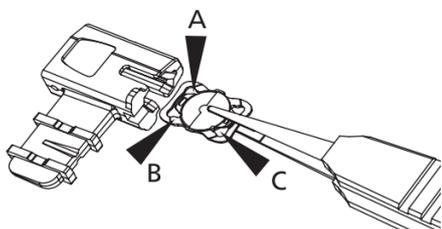

Scan for Video