



# Loading instruction – Viscoject Bio


## EDOF + MIOL LENTIS® with plate-haptic

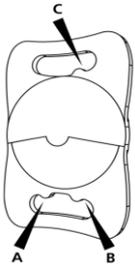

LS-313 MF15  
LS-313 MF20

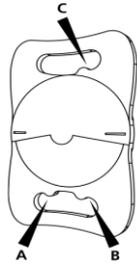


LS-313 MF30  
LS-313 MFX30

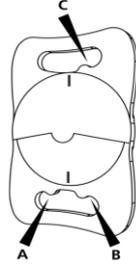

LS-313 MF15 TO-6  
LS-313 MF30 TO-6



LU-313 MF15T/20T/30T/X30T  
LU-313 MF15TY/20TY/30TY/X30TY

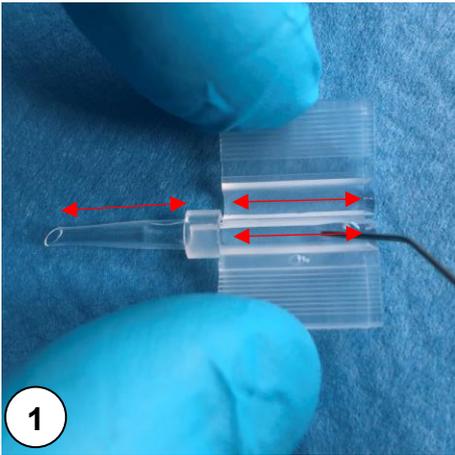

1

Lubricate the injector tip + loading chamber with viscoelastic solution.




2

Open the IOL blister **to your body**.




3

Remove IOL holder, align handle towards your body. **Always keep it horizontal!**

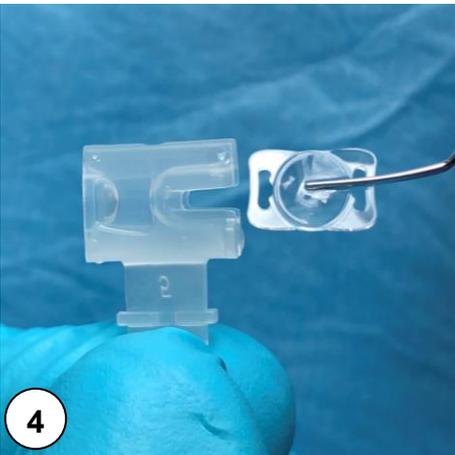


4

**Use a non-toothed forceps** to remove the IOL from the holder without excessive force & keep the **notch face to the right side!**

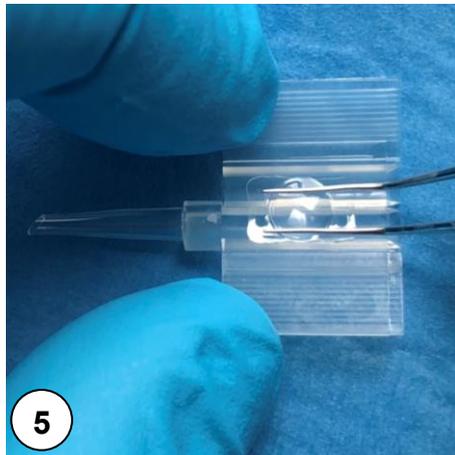



5

Place the IOL within the cartridge, ensure the **correct haptic alignment of A, B and C**. **Push the IOL down** with forceps while closing the cartridge, and a soft **“click”** shall be noticed.

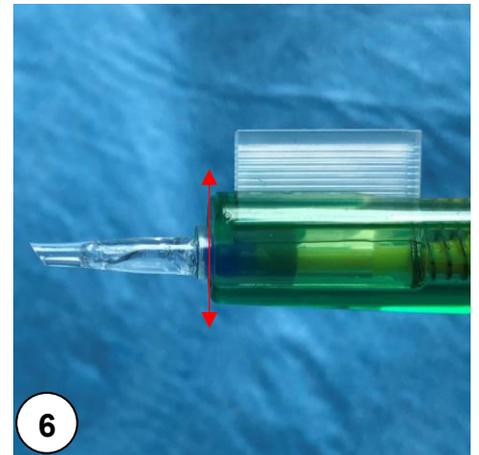

6

Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

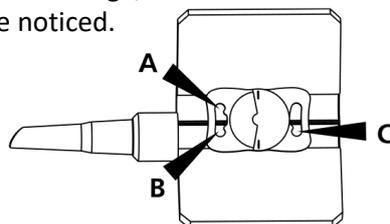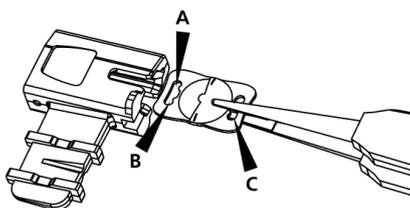

Scan for Video



Teleon Surgical B.V. | Van Rensselaerweg 4 b | 6956AV Spankeren | The Netherlands

www.teleon-surgical.com | iol@teleon-surgical.com

QF2310v04