



Loading instruction - Accuject Back-loaded ACUNEX® Family with C-loop-haptic

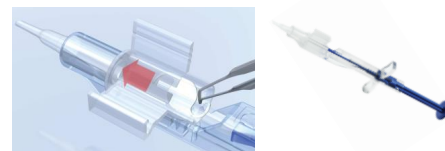


ACUNEX AN6
ACUNEX Quantum AN6Q



ACUNEX Vario AN6V
ACUNEX Variomax AN6VM

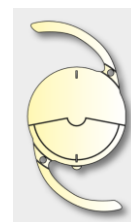


ACUNEX Vario toric AN6V T0-T3
ACUNEX Variomax toric AN6VM T0-T3

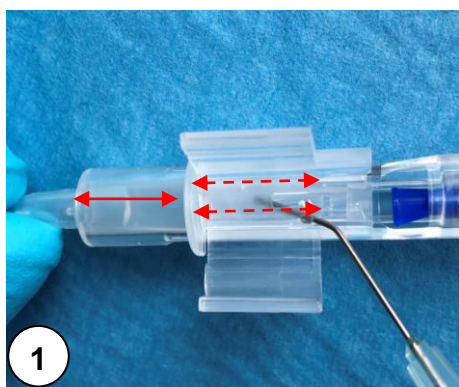


1

Lubricate the injector tip and **underneath** the cover of the loading chamber with viscoelastic solution.

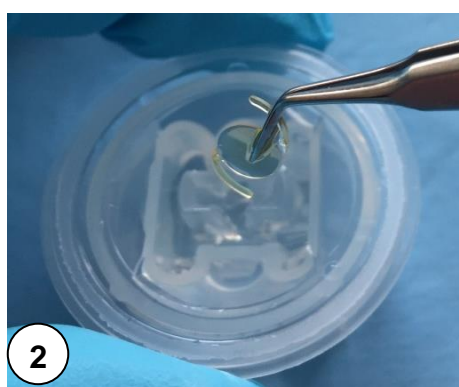


2

Gently remove the IOL with forceps from the IOL holder.
Use a non-toothed forceps!

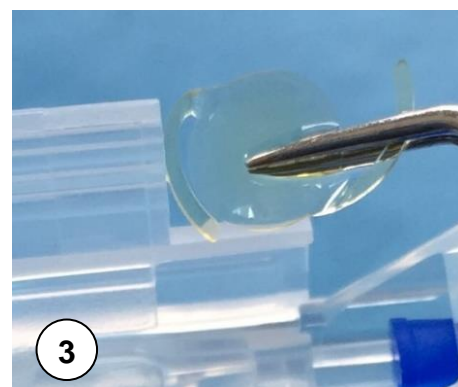


3

Press the leading haptic towards the optic with caution **to ensure the haptic fits closely to the optic during implantation!**

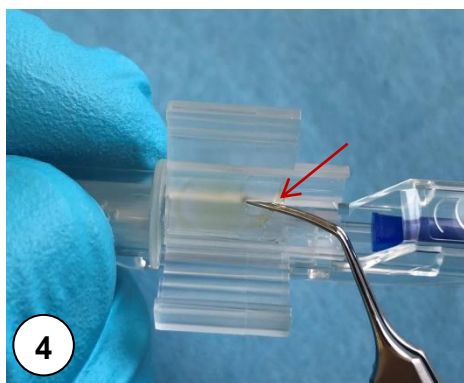


4

Place the IOL underneath the cover. **Push the IOL including rear haptic to the end of guide slot.**

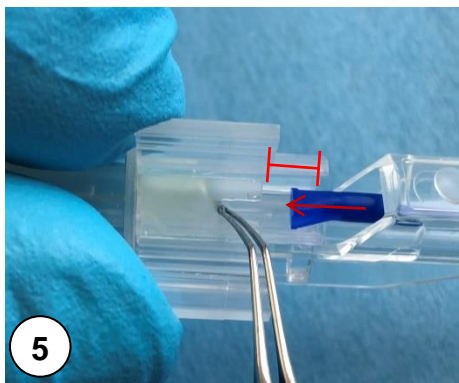

5

Push the plunger forward into the red marked area before closing the wings!

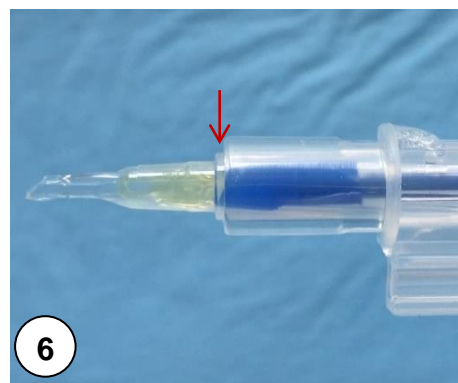

6

Push the plunger until the blue cushion reaches the tip, followed by a short press relief. Implant the IOL.

