

Loading instruction - Accuject Back-loaded ACUNEX® Family with C-loop-haptic




ACUNEX AN6
ACUNEX Quantum AN6Q



ACUNEX Vario AN6V
ACUNEX VarioMax AN6VM




ACUNEX Vario Toric AN6V T0-T3
ACUNEX VarioMax Toric AN6VM T0-T3

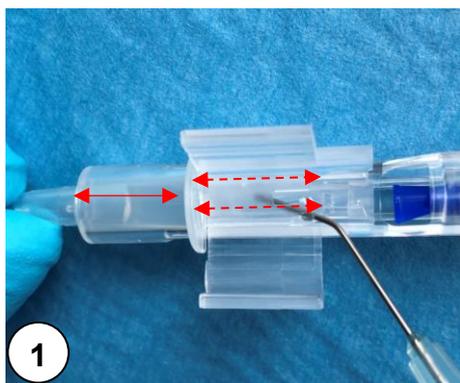


1

Lubricate the injector tip and **underneath** the cover of the loading chamber with viscoelastic solution.

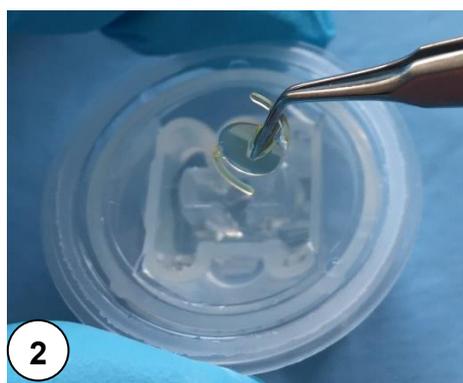


2

Gently remove the IOL with forceps from the IOL holder. **Use a non-toothed forceps!**

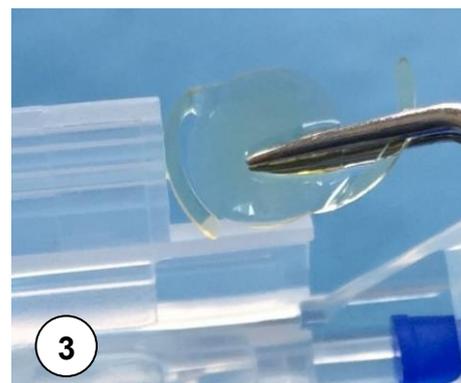


3

Press the leading haptic towards the optic with caution **to ensure the haptic fits closely to the optic during implantation!**

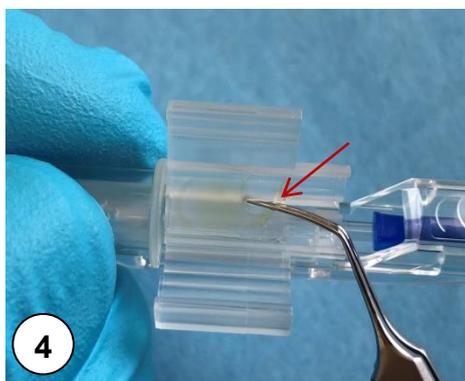


4

Place the IOL underneath the cover, ensure the **correct haptic alignment of A**. **Push the IOL including rear haptic to the end of guide slot.**

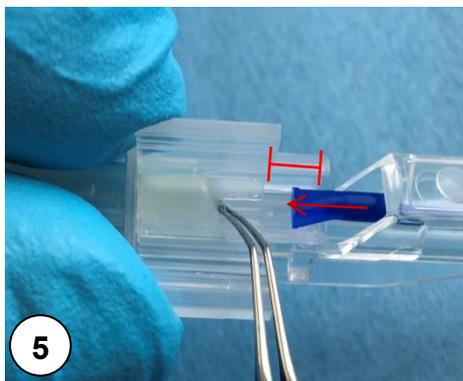



5

Push the plunger forward into the **red marked area** before closing the wings!

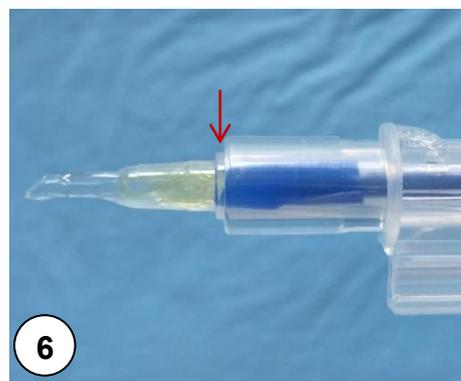

6

Place the cartridge into the injector. Advance the plunger until the blue cushion reaches the tip, **gently retract the plunger**. The IOL is ready for implantation.

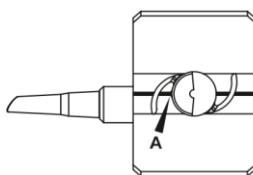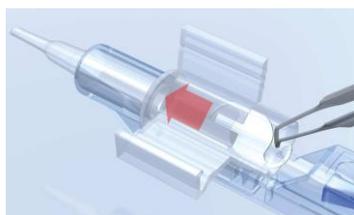

Scan for Video



Teleon Surgical B.V. | Van Rensselaerweg 4 b | 6956AV Spankeren | The Netherlands

www.teleon-surgical.com | iol@teleon-surgical.com

QF2311v04